

SELECTIVE MICRO[®]CLEAN 2L50A

**PRODUCES TWO LITERS OF 50 PPM CHLORINE
DIOXIDE SOLUTION WHEN ACTIVATED**

Intended Use: To clean and remove organic matter from fluid lines and reservoirs in clinical analyzers.

**KEEP OUT OF REACH OF CHILDREN
DANGER**

(See back panel for other cautions)

Active Ingredient: Sodium Chlorite (CAS No. 7758-19-2).....25.5%
Inert Ingredients:..... 74.5%
Total:.....100.0%
Net Weight of contents.....0.07 ounces (1.9 grams)

THIS SIDE UP AFTER FILLING

Amount of Chlorine Dioxide generated = 0.005%

FIRST AID

If in Eyes: Hold eye open and rinse slowly and gently with water for 15 –20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice.

If on Skin or Clothing: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment advice.

If Swallowed: Call poison control center or doctor immediately for treatment advice. Have person sip a glass of water if able to swallow. Do not induce vomiting unless told to do so by the poison control center or doctor. Do not give anything by mouth to an unconscious person.

If Inhaled: Remove victim to fresh air. If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably mouth-to-mouth if possible. Call a poison control center or doctor for treatment advice. Get medical attention.

Have the product container or label with you when calling a poison control center or doctor or going for treatment.

Note to Physician: Probable mucosal damage may contraindicate the use of gastric lavage.

PRECAUTIONARY STATEMENTS

HAZARDS TO HUMANS

DANGER: DRY INGREDIENTS: CORROSIVE. CAUSES IRREVERSIBLE EYE DAMAGE AND CAUSES SKIN BURNS. HARMFUL IF SWALLOWED. WEAR PROTECTIVE EYEWEAR (GOGGLES, FACE SHIELD OR SAFETY GLASSES) WHEN HANDLING DRY INGREDIENTS. WASH THOROUGHLY WITH SOAP AND WATER AFTER HANDLING AND BEFORE EATING, DRINKING OR USING TOBACCO. REMOVE CONTAMINATED CLOTHING AND WASH BEFORE REUSE.

ACTIVATED SOLUTION: CAUSES MODERATE EYE IRRITATION. AVOID CONTACT WITH EYES, SKIN, OR CLOTHING. AVOID BREATHING VAPORS. WASH THOROUGHLY WITH SOAP AND WATER AFTER HANDLING AND BEFORE EATING, DRINKING, CHEWING GUM OR USING TOBACCO.

PHYSICAL OR CHEMICAL HAZARDS

DRY SODIUM CHLORITE IS INCOMPATIBLE WITH ACIDS, REDUCING AGENTS, COMBUSTIBLE MATERIALS, SULFUR-CONTAINING RUBBER, SOLVENTS AND PAINTS. KEEP ACTIVATED SOLUTION FROM LIGHT AND HEAT. CHLORINE DIOXIDE GAS MAY CONCENTRATE IN OPEN SPACE OF POUCH. ALWAYS OPEN ACTIVATED POUCH, AND DILUTE ACTIVATED SOLUTION, IN WELL-VENTILATED AREA.

NOTE: For professional use only by qualified service personnel in the commercial applications discussed below and in accompanying technical information. Not for use in households or other settings where children may be present. Cap is not child-resistant.

DIRECTIONS FOR USE: DO NOT USE THIS PRODUCT IN A MANNER INCONSISTENT WITH THE LABELING.

1. Fill the 2L50A pouch with two (2) liters of distilled or deionized water.
2. Lay filled pouch so that the **Selective Micro[®]Clean** label is in full view on top.
3. WAIT 1 HOUR BEFORE USE TO ENSURE FULL STRENGTH.
4. Shake gently before use.
5. Direct spout away from face when opening, work in a well-ventilated area, and avoid inhaling fumes.
6. Follow specific instrument cleaning protocol to complete procedure.
7. Store unused solution (if any) in pouch in refrigerator or cool place out of direct sunlight. Use solution within 72 hours of generation.
8. Activate prior to expiration date stamped on pouch.

Use in conjunction with *Selective Micro[®] Chlorine Dioxide Test Strips*.

STORAGE AND DISPOSAL

DO NOT CONTAMINATE WATER, FOOD OR FEED BY STORAGE AND DISPOSAL. STORE IN COOL, DRY, VENTILATED AREA. STORE BELOW 50° C (122° F). KEEP PRODUCT OUT OF DIRECT SUNLIGHT. STORE SEPARATELY FROM WATER AND ACIDS. IF PACKAGE RUPTURES AND CONTENTS SPILL, DO NOT PERMIT CONTACT OF CONTENTS WITH ORGANIC MATERIALS (FOR EXAMPLE: CLOTHING OR COMBUSTIBLE MATERIALS) OR ACIDS. COMPLETELY EMPTY ACTIVATED SOLUTION IN POUCH INTO APPLICATION EQUIPMENT. THEN DISPOSE OF EMPTY POUCH IN A SANITARY LANDFILL OR BY INCINERATION, OR, IF ALLOWED BY STATE AND LOCAL AUTHORITIES, BY BURNING. IF BURNED, STAY OUT OF SMOKE.

WARRANTY STATEMENT: The Company warrants the product to be free from defects in material and workmanship. THE COMPANY MAKES NO WARRANTY THAT THE GOODS SHALL BE MERCHANTABILITY. THE COMPANY MAKES NO WARRANTY, EXPRESS OR IMPLIED, EXCEPT SUCH AS IS EXPRESSLY SET FORTH HEREIN. The Company shall not be liable for any incidental or consequential damages for any breach of warranty. The Company's liability for any breach of warranty shall be limited to the purchase price of the product.

Manufactured for:

Selective Micro Technologies
6200 Avery Rd. – Suite A
Dublin, OH 43016

855-256-8299 • www.selectivemicro.com